

## Monteringsbeskrivelse ryggforlenger.

Løsne skruene som vist på bilde 1 og ta ut firkantmutter inne i ryggørret.


Bilde 2 viser ryggforlenger med firkantmutter.

Sett deretter ryggforlengerne ned i ryggørrene og søk med skruen slik at skruen festes i mutter som sitter i ryggforlengerne og stram skruen som vist på bilde 3.

Plasser deretter tekstilet med velcroen på fremsiden slik at den er parallell med velcro på standard ryggen.

Løsne tekstilene fra hverandre og strekk disse slik at de har samme form/bue som velcro på rygg har.

Etter at tekstilet med velcro er innstilt på ønsket bredde så kan den klippes på utsiden av sømmene på hver del.

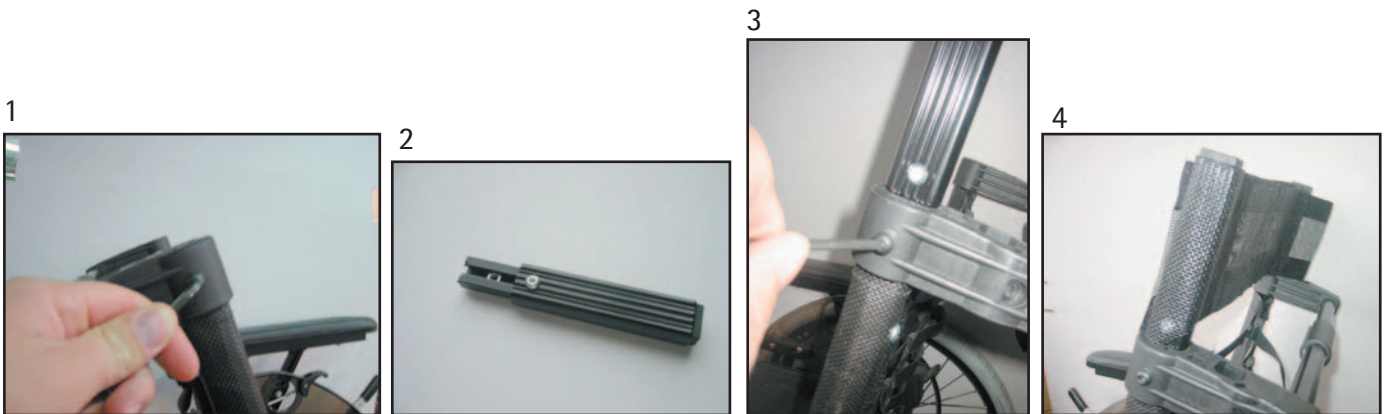

## Mounting description backrest extender.

Unfix the screws as shown on picture 1 and remove the square nut inside the backrest tubes.

Picture 2 shows the backrest extender with the square nut.

Place the backrest extender in the backrest tubes and finish the screw in the square nut of the backrest extender as shown on picture 3.

Place the fabric onto the backrest extenders so that the velcro is equal with the velcro on the standard backrest.

Loosen and place the fabric so that the same shape as the standard backrest before cutting the "long ends" on the outside of the seams.