

Mounting Description Headrest Adaptor Stabilization Unit



1. Remove the screw from the push bar brackets on both sides. (Attention, fix the nut inside of the tube with a tong)
If you are using backheight 60 cm, please replace the original screws with longer screws (M6 X 25 mm)



2. Remove the 4 screws from the head support adaptor.



3. Now place the Headrest Adaptor Stabilization Unit over the push bar brackets and bring the 4 holes in the same position as the 4 holes of the head support adaptor.



4. Fix the Headrest Adaptor Stabilization Unit with the original screws at the push bar bracket.
(Attention, you don't tighten the screws too tightly yet)
If you are using backheight 60 cm, please replace the original screws with longer screws (M6 X 25 mm)



5. Now put the 4 enclosed socket-head cap screw (50 mm long) into the bore-holes and fix the Stabilization Unit with the head support adaptor.
6. At last tighten all the screws and check the solid seat of all your relaxed screws.

