

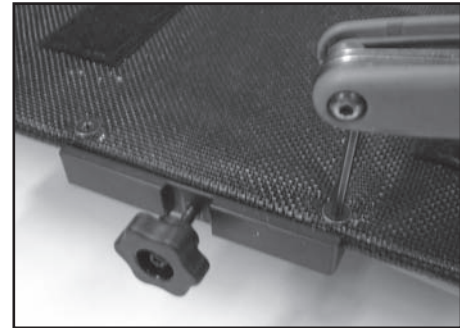


Mounting Description Abduction block

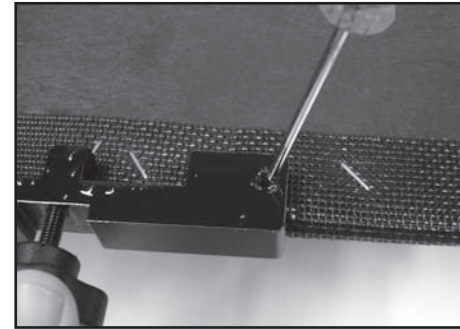

Tools: awl, 6mm drill & 4mm Allen key

1. Place the locking bracket in center of the seat plate.
2. Use an awl to mark the position of the locking bracket screw holes.
The locking bracket is to be mounted on the **UNDERSIDE** of the seat plate.
3. Drill two holes in the seat plate and mount the locking bracket on the underside of the seat plate. Mount it by fixing the screws from the top of the seat plate and down using a 4mm Allen key.
4. Mount the abduction block by placing the adjustment tube into the locking wheel of the bracket. Fix the abduction block by turning the locking wheel.
5. The abduction block can be adjusted 8cm in height (5B) to be compatible with cushions used in the market.
The abduction block can be adjusted 10cm in depth (5A) for individual adjustment.

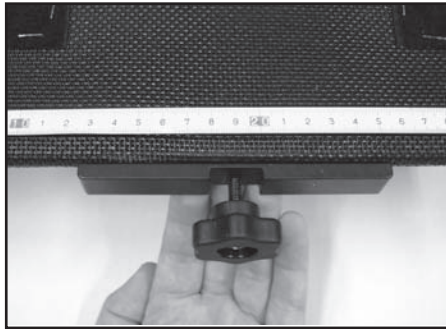

Item number	Item
27490	Abduction block Small 80mm
27491	Abduction block Medium 110mm
27492	Abduction block Large 140mm

