


Mounting description Head support rack Netti Svipp



1. Untighten the adjustment wheel on both sides.

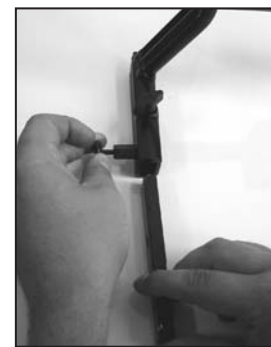

2. Open the spring lock and pull out the locking bar.

3. Remove the plastic plug on top of the backrest tube, and unscrew the screw on the outside of the tube.




4. Place the locking bar into the backrest tube using a plastic hammer, and control it with a tool through the hole in the backrest tube to make sure that the holes are parallel.



5. Fix the locking bolt with the enclosed screw, using a 4mm Allen key.



6. Untighten the screws on the telescope bar using a 5mm Allen key, and pull out to the required width.

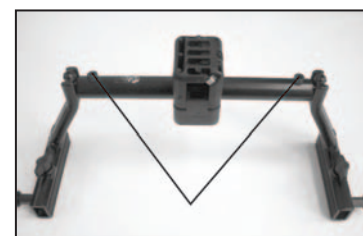

7. Open the spring lock and lead the headrest rack onto the locking bar until you hear it clicks into position.



8. Tighten the adjustment wheel in the rear on both sides.



9



9. Tighten the screws on the telescope bar.

10. Mount the enclosed adjustment lever to the plastic adapter.



10

11



11. Mount the headrest, and make sure that it does not come in conflict with the tie bar on the top of the backrest of the wheelchair.

12. Tighten the screws on the plastic adapter diagonally with a 5mm Allen key.



12

13



13. When the headrest is fitted, tighten the set screw properly with a 3mm Allen key, to avoid further movement.

⚠ Never lift the chair in the headrest rack.

Item number	Item
27214	Headrest rack Netti Svipp 35-45cm cpl
27215	Headrest rack Netti Svipp 50-55cm cpl