


## Mounting description for JAP-Unit

Tools needed: 4mm & 5mm Allen key, 8mm & 10mm open-end spanner & cutting pliers .

To use the JAP unit, the standard anti tip has to be removed.  
The JAP unit will function as anti tip.  
The anti tip is removed by loosening the screw and nut in front on the lower frame tube (use a 4mm Allen key) and pull the anti tip out.  
After removing the anti tip replace the screw and nut.


Screw for releasing anti tip

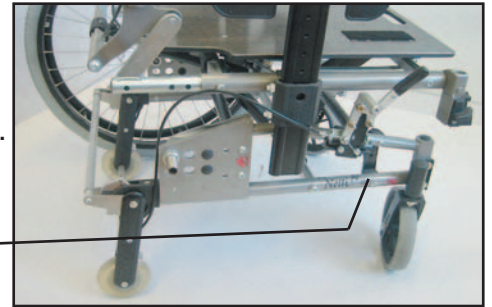

Place the JAP fixing brackets into the holes for the anti tip on the lower frame tube and fix it on both sides with the enclosed screws M6x25 (use 5mm Allen key).



Place the brakes in required position, fix the brake clamp and brake. Pull the wire ( 700mm) and wire stocking (550mm) forward towards the brake and pull the wire through the foot screw (the foot screw should be screwed almost all the way in) and then through the locking screw on the brake. Put the brake in active position (pull the brake backwards), pull the wire hard forward so that the brake on the JAP unit is pressed against the small wheel, tighten the nut (use a 10mm open-end spanner) on the locking screw (tooth washer should be on top of the wire), cut the wire app. 30mm after it's fixed and put on the end sleeve. Adjust the foot screw out and check that the JAP unit is braking properly on both sides.

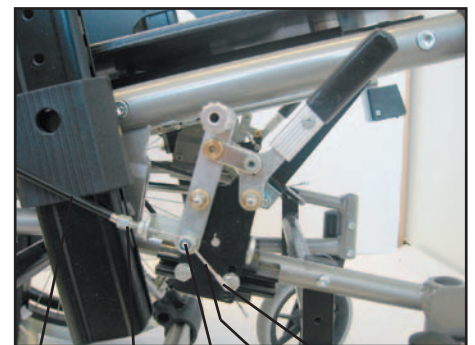

Wire-stocking  
Foot screw  
Nut  
Wire  
End sleeve  
Locking screw

## Mounting description for JAP-Unit

To activate the JAP unit as anti tip going from the position for dismounting of main wheels:  
Push down the upper bar as shown in picture 1A. Pull the lower bar, using the sole of the shoe, backwards/upwards as shown in picture 2A.



To activate the JAP unit for dismounting of the main wheels going from anti tip position:  
Push down the upper bar as shown in picture 1B until the anti tip clicks out of position. Push the lower bar down as shown in picture 2B while lifting the chair by the push bar. Continue pushing the lower bar in under the chair until the JAP unit clicks into position. Picture 3B shows the JAP unit in activated position for dismounting of the main wheels.

**NB:** When the JAP unit is active and the mainwheels are not mounted, never change position of the JAP unit by releasing any of the bars. This will make the wheelchair fall backwards and the user might get hurt.



To place the JAP unit in horizontal position for passing of curbs etc., lift the upper bar with the left hand and pull the lower bar backwards/upwards using the right hand until the JAP unit clicks into position (as shown in picture 1C). To reactivate the JAP unit as anti tip, push the lower bar straight down as shown in picture 3C until it clicks into position. Picture 2C shows the JAP unit in horizontal position.

